



COBAS AmpliScreen™ HCV Test, version 2.0

FOR *IN VITRO* DIAGNOSTIC USE.

COBAS AmpliScreen HCV Test, version 2.0	<input type="text" value="HCV"/>	48 Tests	P/N: 03139204 123
COBAS AmpliScreen Multiprep Specimen Preparation Reagents Kit	<input type="text" value="MULTIPREP"/>	48 Tests	P/N: 03184498 123
COBAS AmpliScreen Negative Human Plasma Reagent Kit	<input type="text" value="NHP"/>	96 Tests	P/N: 03184480 123
COBAS AMPLICOR™ Wash Buffer	<input type="text" value="WB"/>	500 Tests	P/N: 20759899 123 ART: 07 5989 9 US: 83314

INTENDED USE

The COBAS AmpliScreen™ HCV Test, version 2.0 (v2.0) is a qualitative *in vitro* test for the direct detection of Hepatitis C Virus (HCV) RNA in human plasma from donations of whole blood and blood components for transfusion.

The test is intended for use in screening individual donor samples of human plasma, or pools of human plasma comprised of equal aliquots of not more than 24 individual donations. The test is intended to be used for detecting HCV RNA in conjunction with licensed tests for detecting antibodies to HCV.

This assay is not intended for use as an aid in diagnosis.

SUMMARY AND EXPLANATION OF THE TEST

Hepatitis C Virus is considered to be the principal etiologic agent responsible for 90-95% of the cases of post-transfusion non-A and non-B hepatitis.^{1,2} HCV is a single-stranded, positive sense RNA virus with a genome of approximately 10,000 nucleotides coding for 3,000 amino acids.¹ As a blood-borne virus, HCV can be transmitted by blood and blood products. The global prevalence of HCV infection, as determined by immunoserology, ranges from 0.6% in Canada to 1.5% in Japan.²

Serological screening assays have greatly reduced, but not completely eliminated, the risk of transmitting viral infections by transfusion of blood products.³⁻⁶ Recent studies indicate that nucleic acid-based amplification tests for HCV RNA will allow detection of HCV infection earlier than the current antibody based tests. Nucleic acid testing (NAT) of whole blood donations has been in place in the United States since 1999 under Investigational New Drug Application (IND). Nucleic acid-based tests can detect viremic units donated by carriers who do not seroconvert or who lack antibodies to serological markers normally detected by immunological assays.⁷⁻⁹

The COBAS AmpliScreen HCV Test, v2.0, uses a generic sample preparation technique in a mini-pool testing format along with automated amplification and detection using PCR on the COBAS AMPLICOR™ Analyzer for the detection of HCV RNA in blood donations. The assay incorporates an Internal Control for monitoring assay performance in each individual test as well as the AmpErase® (uracil-N-glycosylase) enzyme to reduce potential contamination by previously amplified material (amplicon).

PRINCIPLES OF THE PROCEDURE

The COBAS AmpliScreen HCV Test, v2.0 is based on five major processes:

1. Sample Processing
2. Reverse transcription of target RNA to generate complementary DNA (cDNA)¹⁰
3. PCR amplification¹⁰ of target cDNA using HCV-specific complementary primers
4. Hybridization of the amplified products to oligonucleotide probes specific to the target(s)
5. Detection of the probe-bound amplified products by colorimetric determination

Sample Processing

Two specimen processing procedures are used with the COBAS AmpliScreen HCV Test, v2.0 as follows:

- Multiprep Specimen Processing Procedure for preparation of mini-pool specimens
- Standard Sample Processing Procedure for preparation of individual donor samples

In the Standard Specimen Processing Procedure, HCV RNA is isolated directly from plasma by lysis of the virus particles with Multiprep Lysis Reagent followed by precipitation of the RNA with alcohol. In the Multiprep Specimen Processing Procedure, HCV viral particles are first pelleted from the plasma sample by high speed centrifugation, followed by lysis of the pelleted virus with a chaotropic agent (Multiprep Lysis Reagent) and precipitation of the RNA with alcohol.

The Multiprep Internal Control (**MP IC**), containing the HCV Internal Control, is introduced into each sample with the Multiprep Lysis Reagent and serves as an extraction and amplification control for each processed specimen and control. The HCV Internal Control is an RNA transcript with primer binding regions identical to those of the HCV target sequence, a randomized internal sequence of similar length and base composition as the HCV target sequence, and a unique probe binding region that differentiates the HCV Internal Control amplicon from target amplicon. These features were selected to ensure equivalent amplification of the HCV Internal Control and the HCV target RNA.

Reverse Transcription

The reverse transcription and amplification reactions are performed with the thermostable recombinant enzyme *Thermus thermophilus* DNA Polymerase (*rTth* pol). In the presence of manganese (Mn^{2+}) and under the appropriate buffer conditions, *rTth* pol has both reverse transcriptase and DNA polymerase activity.¹⁰ This allows both reverse transcription and PCR amplification to occur in the same reaction mixture. Reverse transcription using *rTth* pol produces a cDNA copy of the HCV target and the HCV Internal Control RNA.

PCR Amplification

Following reverse transcription using *rTth* pol, a second DNA strand is produced from the cDNA copy, thereby yielding a double-stranded DNA copy of the HCV target and HCV Internal Control RNA. The reaction mixture is heated to separate the resulting double-stranded DNA. As the mixture cools, primers anneal to the target DNA and in the presence of Mn^{2+} and excess deoxynucleotide triphosphates (dNTPs), the *rTth* pol extends the annealed primers along the target templates to produce a double-stranded DNA molecule termed an amplicon. The COBAS AMPLICOR Analyzer automatically repeats this process for a designated number of cycles, each cycle effectively doubling the amount of amplicon DNA. The required number of cycles is preprogrammed in the COBAS AMPLICOR Analyzer.

Selective Amplification

To ensure selective amplification of nucleic acid target in the sample and prevent amplification of pre-existing amplicon, AmpErase® (uracil-N-glycosylase) enzyme is added to the COBAS AmpliScreen HCV Test, v2.0. The AmpErase enzyme recognizes and catalyzes the destruction of DNA strands containing deoxyuridine¹¹, but not DNA containing deoxythymidine. Deoxyuridine is not present in naturally occurring DNA, but is always present in amplicon because of the use of deoxyuridine triphosphate in place of deoxythymidine triphosphate as one of the dNTPs in the Master Mix reagent; therefore, only amplicon contain deoxyuridine. Deoxyuridine renders contaminating amplicon susceptible to destruction by the AmpErase enzyme before amplification of the target DNA. The AmpErase enzyme, which is included in the Master Mix reagent, catalyzes the cleavage of DNA, thereby rendering the DNA non-amplifiable. The AmpErase enzyme is inactive at temperatures above 55°C, i.e., throughout the thermal cycling steps, and therefore does not destroy target amplicon. Following amplification, any residual enzyme is denatured by the addition of the Denaturation Solution, thereby preventing the degradation of any target amplicon.

Hybridization Reaction

Following PCR amplification, the COBAS AMPLICOR Analyzer automatically adds Denaturation Solution to the A-tubes to chemically denature the HCV amplicon and the HCV Internal Control amplicon to form single-stranded DNA. Aliquots of denatured amplicon are then transferred to two detection cups (D-cups). A suspension of magnetic particles coated with an oligonucleotide probe specific for HCV amplicon or HCV Internal Control amplicon is added to the individual D-cups. The biotin-labeled HCV target and HCV Internal Control amplicon are hybridized to the target-specific oligonucleotide probes bound to the magnetic particles. This hybridization of amplicon to the target-specific probe increases the overall specificity of the test.

Detection Reaction

Following the hybridization reaction, the COBAS AMPLICOR Analyzer washes the magnetic particles in the D-cups to remove unbound material, and then adds avidin-horseradish peroxidase conjugate. The avidin-horseradish peroxidase conjugate binds to the hybridized biotin-labeled amplicon. The COBAS AMPLICOR Analyzer removes unbound conjugate by washing the magnetic particles and then adds a substrate solution containing hydrogen peroxide and 3,3',5,5'-tetramethylbenzidine (TMB) to each D-cup. In the presence of hydrogen peroxide, the particle-bound horseradish peroxidase catalyzes the oxidation of TMB to form a colored complex. The absorbance is measured by the COBAS AMPLICOR Analyzer at a wavelength of 660 nm.

MATERIALS PROVIDED BY ROCHE

The COBAS AmpliScreen Multiprep Specimen Preparation Reagents Kit, COBAS AmpliScreen Negative Human Plasma Reagent Kit and the COBAS AMPLICOR Wash Buffer are provided as stand-alone kits to be used in conjunction with the COBAS AmpliScreen HCV Test, v2.0, as well as with the COBAS AmpliScreen™ HIV-1 Test, v1.5.

COBAS AmpliScreen Multiprep Specimen Preparation Reagents Kit (P/N: 03184498 123)	MULTIPREP	48 Tests
MP (-) C [Multiprep (-) Control]		
MP LYS (Multiprep Lysis Reagent)		
MP DIL (Multiprep Specimen Diluent)		
MP IC (Multiprep Internal Control)		
COBAS AmpliScreen Negative Human Plasma Reagent Kit (P/N: 03184480 123)	NHP	96 Tests
NHP [Negative Plasma (Human)]		
COBAS AmpliScreen HCV Test, version 2.0 (P/N: 3139204 123)	HCV	48 Tests
COBAS AmpliScreen HCV Positive Control Reagent, version 2.0	HCV CTL	
HCV (+) C, v2.0 [HCV (+) Control, version 2.0]		
COBAS AmpliScreen HCV Amplification Reagents, version 2.0	HCV AMP	
HCV MMX, v2.0 (HCV Master Mix, version 2.0)		
HCV Mn²⁺, v2.0 (HCV Manganese Solution, version 2.0)		
COBAS AmpliScreen HCV Detection Reagents, version 2.0	HCV DK	
CH PS1, v2.0 (HCV Probe Suspension 1, version 2.0)		
CH2, v2.0 (HCV Probe Suspension 2, version 2.0)		
CI PS1 (IC Probe Suspension 1)		
CI2 (IC Probe Suspension 2)		
DN2 (Denaturation Solution)		
CN4 (Avidin - Horseradish Peroxidase Conjugate)		
SB1 (Substrate A)		
SB (Substrate B)		
COBAS AMPLICOR Wash Buffer (P/N: 20759899 123; ART: 07 5989 9; US: 83314)	WB	500 Tests
WB (10X-Wash Concentrate)		

OTHER MATERIALS REQUIRED BUT SOLD SEPARATELY (MAY BE PURCHASED FROM ROCHE)

- COBAS AMPLICOR Analyzer, Printer, and *Operator's Manual* for the COBAS AMPLICOR Analyzer
- COBAS AMPLICOR A-rings
- COBAS AMPLICOR D-cups
- AMPLILINK® Software and Operator's Manual for the AMPLILINK software
- Sarstedt 1.5-mL tube Barcode Labels
- Hamilton Archive and Intermediate Plate Barcode Labels

MATERIALS REQUIRED BUT NOT PROVIDED BY ROCHE

- Hamilton MICROLAB® AT plus 2 Pipettor or equivalent
- Refrigerated high speed centrifuge with fixed angle rotor (45 degrees, capacity for at least 24 x 1.5-mL tubes) with an RCF of 23,600 x g (Heraeus Centrifuge 17RS or Biofuge 28RS with HFA 22.1 rotor, Heraeus Biofuge Stratos with the 3331 rotor or equivalent).
- Microcentrifuge, (max. RCF 16,000 x g, min. RCF 12,500 x g) (Eppendorf® 5415C, HERMLE Z230M, or equivalent)
- Eppendorf 1.25 mL Eppendorf Combitip® Reservoir (sterile) or equivalent
- Eppendorf Multipipette® pipette or equivalent
- Ethanol, 90% or 95%, reagent grade for Molecular Biology or Histology use
- Distilled or deionized water
- Powderless, disposable gloves
- Isopropyl alcohol, reagent grade

- Disposable, Sterile, Polystyrene pipettes (5 mL, 10 mL and 25 mL)
- Sterile, RNase-free, fine-tip transfer pipettes
- Pipettors (capacity 20 μ L to 1000 μ L, capable of providing \pm 3% accuracy and precision \leq 5%) with aerosol barrier or positive displacement RNase-free tips
- Tube racks (Sarstedt P/N 93.1428 or equivalent)
- 1.5-mL sterile, non-siliconized, conical polypropylene screw-cap tubes, (Sarstedt 72.692.105 or equivalent)
- Vortex mixer
- Hamilton Slotted Deepwell Archive Plate, 2.2 mL and Sealing Capmat
- Hamilton Slotted Intermediate Plate

REAGENTS**COBAS AmpliScreen Multiprep Specimen Preparation Reagents Kit**

MULTIPREP

48 Tests**MP (-) C**

[Multiprep (-) Control]

6 x 0.1 mL


- < 0.005% Poly rA RNA (synthetic)
- EDTA
- 0.05% Sodium azide

MP LYS

(Multiprep Lysis Reagent)

6 x 9.0 mL

- Tris-HCl buffer
- 60% Guanidine thiocyanate
- 3% Dithiothreitol
- < 1% Glycogen

Xn  60% (w/w) Guanidine thiocyanate

Harmful

MP DIL

(Multiprep Specimen Diluent)

6 x 4.8 mL

- Tris-HCl buffer
- < 0.005% Poly rA RNA (synthetic)
- EDTA
- 0.05% Sodium azide

MP IC

(Multiprep Internal Control)

6 x 0.1 mL

- Tris-HCl buffer
- < 0.001% Non-infectious plasmid DNA containing HBV primer binding sequences and a unique probe binding region
- < 0.001% Non-infectious *in vitro* transcribed RNA (microbial) containing HCV primer binding sequences and a unique probe binding region
- < 0.001% Non-infectious *in vitro* transcribed RNA (microbial) containing HIV-1 primer binding sequences and a unique probe binding region
- < 0.005% Poly rA RNA (synthetic)
- EDTA
- < 0.1% Amaranth dye
- 0.05% Sodium azide

COBAS AmpliScreen Negative Human Plasma Reagent Kit

NHP

48 Tests**NHP**

[Negative Plasma (Human)]

24 x 1.6 mL

- Human plasma, non-reactive by US FDA licensed tests for antibody to HCV, antibody to HIV-1/2, HIV p24 antigen and HBsAg. HIV-1 RNA, HCV RNA and HBV DNA not detectable by PCR methods.
- 0.1% ProClin[®] 300

COBAS AmpliScreen HCV Test, version 2.0

HCV

48 Tests**COBAS AmpliScreen HCV Positive Control Reagent, version 2.0**

HCV CTL

6 x 0.1 mL

HCV (+) C, v2.0

[HCV (+) Control, version 2.0]

- < 0.001% Non-infectious *in vitro* transcribed RNA (microbial) containing HCV sequences
- < 0.005% Poly rA RNA (synthetic)
- EDTA
- 0.05% Sodium azide

COBAS AmpliScreen HCV Amplification Reagents

HCV AMP

6 x 0.7 mL

HCV MMX, v2.0

(HCV Master Mix, version 2.0)

- Bicine buffer
- 16% DMSO
- Glycerol
- < 0.01% *rTth* DNA Polymerase (*rTth* pol, microbial)
- Potassium acetate
- < 0.001% dATP, dCTP, dGTP, dUTP
- < 0.005% KY78 and KY80 primers (KY78 is biotinylated)
- < 0.01% AmpErase[®] (uracil-N-glycosylase) enzyme (microbial)
- 0.05% Sodium azide

HCV Mn²⁺, v2.0

(HCV Manganese Solution, version 2.0)

6 x 0.1 mL

- < 2% Manganese
- Acetic acid
- Amaranth dye
- 0.05% Sodium azide

COBAS AmpliScreen HCV Detection Reagents, version 2.0

HCV DK

1 x 50 Tests

CH PS1 v2.0

(HCV Probe Suspension 1, version 2.0)


- MES buffer
- < 0.4% Suspension of Dynabeads[®] (paramagnetic particles) coated with HCV-specific oligonucleotide capture probe KY150
- 0.09% Sodium azide

CH2, v2.0

(HCV Probe Suspension 2, version 2.0)

1 x 50 Tests

Sodium phosphate buffer
34.7% Sodium thiocyanate
0.2% Solubilizer

Xn  34.7% (w/w) Sodium thiocyanate

Harmful

CI PS1

(IC Probe Suspension 1)

1 x 50 Tests

MES buffer
< 0.4% Suspension of Dynabeads (paramagnetic particles) coated with IC-specific oligonucleotide capture probe SK535
0.09% Sodium azide

CI2

(IC Probe Suspension 2)

1 x 50 Tests


Sodium phosphate buffer
24.9% Sodium thiocyanate
0.2% Solubilizer

DN2

(Denaturation Solution)

1 x 50 Tests

1.6% Sodium hydroxide
EDTA
Thymol blue

Xi  1.6% (w/w) Sodium hydroxide

Irritant

CN4

(Avidin-Horseradish Peroxidase Conjugate)

1 x 100 Tests

Tris-HCl buffer
< 0.001% Avidin-horseradish peroxidase conjugate
Bovine serum albumin (mammalian)
Emulsit 25 (Dai-ichi Kogyo Seiyaku Co., Ltd.)
0.1% Phenol
1% ProClin® 150

SB1

(Substrate A)

6 x 25 Tests


Citrate solution
0.01% Hydrogen peroxide
0.1% ProClin 150

SB

(Substrate B)

6 x 25 Tests
(6 x 5 mL)

0.1% 3,3',5,5'-Tetramethylbenzidine (TMB)
40% Dimethylformamide (DMF)

T  40% (w/w) Dimethylformamide (DMF)

Toxic

R: 61-20/21-36 May cause harm to the unborn child. Harmful by inhalation and in contact with skin. Irritating to eyes.

S: 53-45 Avoid exposure - obtain special instructions before use. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

COBAS AMPLICOR Wash Buffer

WB

500 Tests

WB

(10X-Wash Concentrate)

2 x 250 Tests

< 2% Phosphate buffer
< 9% Sodium chloride
EDTA
< 2% Detergent
0.5% ProClin® 300

STORAGE INSTRUCTIONS

- Room Temperature is defined as 15 - 30°C.
- Do not freeze reagents.
- Store the following reagents at 2 - 8°C. Unopened, these reagents are stable until the expiration date indicated.

MP LYS, MP IC, HCV (+) C, v2.0; MP (-) C, MP DIL and NHP**HCV MMX, v2.0 and HCV Mn²⁺, v2.0****CH PS1, v2.0, CH2, v2.0, CI PS1 and CI2****CN4, SB1 and SB**

- Store **DN2** at 2 - 25°C. Store **WB** at 2 - 30°C. **DN2** and **WB** are stable until the expiration dates indicated.
- Do not expose **SB1, SB** or Working Substrate to metals, oxidizing agents or direct sunlight.
- The following reagents are one time use. Discard any unused portion.

MP IC, HCV (+) C, v2.0, MP (-) C, MP DIL and NHP**HCV Mn²⁺, v2.0, SB, CH PS1, v2.0 and CI PS1.**

PRECAUTIONS**FOR IN VITRO DIAGNOSTIC USE.**

- A. Specimens may be infectious. Use Universal Precautions when performing the assay.¹²⁻¹³ Only personnel proficient in the use of the COBAS AmpliScreen Test System and trained in handling infectious materials should perform this procedure. Thoroughly clean and disinfect all work surfaces with a freshly prepared solution of 0.5% sodium hypochlorite in distilled or deionized water. Follow by wiping down the surface with 70% ethanol.
- B. **CAUTION: The Negative Human Plasma (NHP) of this kit contains human blood products non-reactive by US FDA licensed tests for antibody to HIV-1/2, antibody to HCV, HIV-1 p24 antigen and HBsAg. Testing of Negative Human Plasma by PCR methods showed no detectable HIV-1 RNA, HCV RNA or HBV DNA. No known test method can offer complete assurance that products derived from human blood will not transmit infectious agents. All human blood-sourced materials should be considered potentially infectious and should be handled with Universal Precautions.** If spillage occurs, immediately disinfect, then wipe up with a 0.5% (final concentration) sodium hypochlorite solution (diluted bleach) or follow appropriate site procedures.
- C. Use routine laboratory precautions. Do not pipette by mouth. Do not eat, drink or smoke in designated work areas. Wear disposable gloves, laboratory coats and eye protection when handling specimens and kit reagents. Wash hands thoroughly after handling specimens and kit reagents.
- D. This product contains sodium azide as a preservative. Do not use metal tubing for reagent transfer. If solutions containing azide compounds are disposed of in a plumbing system, they should be diluted and flushed with generous amounts of running water. These precautions are recommended to avoid accumulation of deposits in metal piping in which explosive conditions could develop.
- E. **Heparin has been shown to inhibit PCR. Do not use heparinized plasma with this procedure.**
- F. Use only supplied or specified required disposables to ensure optimal assay performance.
- G. Screw-cap tubes must be used for specimen and control preparation to prevent splashing and potential cross-contamination of specimens and controls. **Do not use snap cap tubes.**
- H. Adequately vortex, where specified, to ensure optimal assay performance.
- I. Handle all materials containing specimens or controls according to Good Laboratory Practices in order to prevent cross-contamination of specimens or controls.
- J. Before use, visually inspect each reagent bottle to ensure that there are no signs of leakage and/or abnormal color. If there is any evidence of leakage and/or abnormal color, do not use that bottle for testing.
- K. Dispose of all materials that have come in contact with specimens and reagents in accordance with country, federal, state and local regulations.
- L. Do not use a kit after its expiration date. DO NOT interchange, mix, or combine reagents from kits with different master lot numbers. Do not use expired reagents.
- M. Material Safety Data Sheets (MSDS) are available on request.
- N. Supplies and equipment must be dedicated to each pre-amplification activity and should not be used for other activities or moved between areas. **Fresh, clean gloves must be worn in each area and must be changed before leaving that area.** Equipment and supplies used for reagent preparation must not be used for specimen preparation activities or for pipetting or processing amplified DNA or other sources of target DNA. Post-amplification supplies and equipment must remain in the Post-Amplification Area at all times.
- O. Avoid contact of **MP LYS, HCV MMX, v2.0, HCV Mn²⁺, v2.0, CH2, v2.0, CI2, DN2, CN4, SB1, SB** and Working Substrate (mixed **SB1** and **SB** reagent) with the skin, eyes or mucous membranes. **If contact does occur, immediately wash with large amounts of water, otherwise burns can occur.** If these reagents are spilled, dilute with water before wiping dry. **Do not allow MP LYS, which contains guanidine thiocyanate, or CH2, v2.0 and CI2, which contain sodium thiocyanate, to contact sodium hypochlorite (bleach) solution. This mixture can produce a highly toxic gas.**
- P. **SB and Working Substrate contain dimethylformamide, which has been reported to be toxic in high oral doses and may be harmful to the unborn child. Skin contact, inhalation of fumes and ingestion should be avoided. If skin contact occurs, wash thoroughly with soap and water and seek medical advice immediately.**
- Q. Refer to the Pooling System Guide for use with the COBAS AmpliScreen Tests and the *Operator's Manuals* for the AMPLILINK software and COBAS AMPLICOR Analyzer.
- R. Closely follow procedures and guidelines provided to ensure that the specimen and control preparation is performed correctly. Any deviation from the given procedures and guidelines may affect optimal assay performance.

REAGENT PREPARATION

- A. **MP IC, HCV (+) C, v2.0; MP (-) C, MP DIL and NHP**
1. Warm **MP IC, HCV (+) C, v2.0; MP (-) C, MP DIL and NHP** to room temperature before use by using a 37°C incubator or on the laboratory bench top.
- B. **Working Lysis Reagent**
1. Warm **MP LYS** to 25 - 37°C to dissolve precipitate (maximum 30 minutes). Mix thoroughly until the crystals are dissolved. Prior to use, examine each bottle of **MP LYS** against a white background for appearance of a yellow color or signs of leakage. If there is any yellow color or signs of leakage do not use that bottle for testing. Contact your local Roche office for replacement.
 2. Vortex **MP IC** briefly before use. Tap vial to collect the solution in the base. Pipette 100 µL **MP IC** into 1 bottle **MP LYS**. Cap the **MP LYS** bottle and vortex briefly. The pink color confirms that the **MP IC** has been added to the **MP LYS**. Discard the remaining **MP IC**.
 3. Store Working Lysis Reagent at room temperature. Use within 4 hours of preparation.
- C. **Working Amplification Master Mix**
1. Prepare Working Master Mix in a template-free area (e.g., in a dead air box). **Reagent preparation area must be clean and disinfected in accordance with methods outlined in Precautions (Item A). Failure to do so may result in reagent contamination.**
 2. Pipette 100 µL **HCV Mn²⁺, v2.0** into 1 bottle **HCV MMX, v2.0**. Recap **HCV MMX, v2.0** bottle and mix well by inverting 10-15 times. The pink color confirms that the **HCV Mn²⁺, v2.0** has been added to the **HCV MMX, v2.0**. Discard the remaining **HCV Mn²⁺, v2.0**. Do not vortex the Working Master Mix. These reagents do not need to be at room temperature before use.
 3. Store at 2 - 8°C and use within 4 hours of preparation.
- D. **Working Probe Suspension Detection Reagents**
1. Prepare Working HCV Probe Suspension: Mix **CH PS1, v2.0** well by vortexing briefly to suspend the microparticles. Pipette 2.5 mL **CH PS1, v2.0** into one **CH2, v2.0** cassette.
 2. Prepare Working IC Probe Suspension: Mix **CI PS1** well by vortexing briefly to suspend the microparticles. Pipette 2.5 mL **CI PS1** into one **CI2** cassette.
 3. Both Working Probe Suspension Detection Reagents are stable for 30 days at 2 - 8°C. Working Reagents can be used for a maximum of six instrument cycles (12 hours per cycle). Mixing occurs automatically on the COBAS AMPLICOR Analyzer.
 4. Store Working Probe Suspension Detection Reagents at 2 - 8°C between instrument cycles. Remove from refrigerator 30 minutes before use on the COBAS AMPLICOR Analyzer.
- E. **DN2 - Denaturation Reagent and CN4 Conjugate Reagent**
1. Once opened, **DN2** and **CN4** are stable for 30 days at 2 - 8°C, or until the expiration date, whichever comes first. Both **DN2** and **CN4** can be used for a maximum of six instrument cycles (12 hours per cycle).
 2. Store **DN2** and **CN4** at 2 - 8°C between instrument cycles. Remove from refrigerator 30 minutes before use on the COBAS AMPLICOR Analyzer.
- F. **Working Substrate Reagent**
1. Working Substrate must be prepared each day by pipetting 5 mL **SB** into one **SB1** cassette. Pipette up and down at least 5 times to mix.
 2. Working Substrate is stable on the COBAS AMPLICOR Analyzer for a maximum of 16 hours.
 3. Do not expose **SB1, SB** or Working Substrate to metals, oxidizing agents, or direct light.

G. Wash Buffer Reagent

1. Examine **WB** before dilution and if necessary, warm at 30 - 37°C to dissolve any precipitate. Add 1 volume of **WB** to 9 volumes of distilled or deionized water. Mix well. Keep a minimum of 3 - 4 liters of Working Wash Buffer (1X) in the Wash Buffer Reservoir of the COBAS AMPLICOR Analyzer at all times.
2. Working Wash Buffer (1X) should be stored at 2 - 25°C in the COBAS AMPLICOR Wash Buffer Reservoir and is stable for 2 weeks from the date of preparation.


H. 70% Ethanol

1. Prepare 70% ethanol fresh daily.
2. One mL 70% ethanol is needed for each specimen and control processed. For example, mix 11.7 mL 90% ethanol and 3.3 mL of distilled or deionized water for every 12 specimens and controls to be processed.

SPECIMEN COLLECTION, STORAGE AND POOLING

NOTE: Handle all specimens as if they are potentially infectious agents.

- A. EDTA, CPD, CPDA-1, CP2D, ACD-A and 4% Sodium Citrate may be used with the COBAS AmpliScreen HCV Test, v2.0. Follow sample tube manufacturer's instructions.
- B. Blood collected in EDTA may be stored at 2 - 30°C for up to 72 hours from time of draw, followed by an additional two days at 2 - 8°C. For storage longer than five days, remove the plasma from the red blood cells by centrifugation at 800 - 1600 x g for 20 minutes. Following removal, plasma may be stored at 2 - 8°C for an additional seven days. Alternatively, plasma may be stored at ≤ -18°C for up to one month.

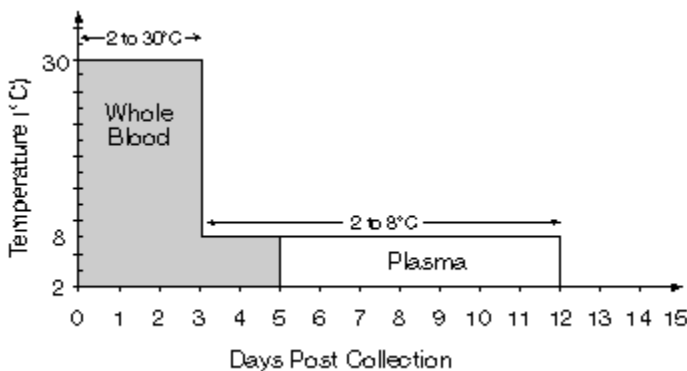

- C. Blood collected in CPD, CPDA-1, or CP2D may be stored for up to 72 hours at 1 - 24°C. Following centrifugation of the CPD, CPDA-1, or CP2D samples at 800-1600 x g for 20 minutes, plasma may be stored at 1 - 6°C for an additional 7 days from the date the plasma was removed from the red blood cells. Plasma separated from the cells may be stored at ≤ -18°C for up to one month.
- D. ACD-A or 4% sodium citrate anticoagulated apheresis plasma can be stored at 1 - 6°C for up to 6 hours, followed by subsequent storage at ≤ -18°C for up to one month.
- E. Do not freeze whole blood.
- F. **Heparin has been shown to inhibit PCR. Use of heparinized specimens is not recommended.**
- G. Warm pooled or individual donor specimens to room temperature before using.
- H. Covered Archive Plates may be stored at 2 - 8°C for up to 7 days from the date the plasma was removed from the red blood cells.
- I. No adverse effect on assay performance was observed when plasma specimens were subjected to three freeze-thaw cycles.
- J. Thaw frozen specimens at room temperature before using.
- K. The user should validate other collection and storage conditions. If specimens are to be shipped, they should be packaged and labeled in compliance with applicable federal and international regulations covering the transport of clinical specimens and etiologic agents.¹⁴
- L. **False positive results may occur if cross contamination of specimens is not adequately controlled during specimen handling and processing.**
- M. **SPECIMEN POOLING:**
 1. The Pooling System for use with the COBAS AmpliScreen Tests performs barcode scanning and pooling operations that combine aliquots from 24 individual samples into a single Primary Pool that is used for testing. The pooling algorithm requires preparation of Secondary Pools as well as individual specimens for follow-up testing in the event a Primary Pool tests positive. If less than 24 specimens are available, testing is performed using the individual specimens.

NOTE: The user must validate other pooling algorithms and equipment.

PROCEDURAL NOTES**A. Run Size**

1. Each kit contains reagents sufficient for six 8-specimen runs, which may be performed separately or simultaneously. At least one preparation of the COBAS AmpliScreen Multiprep Negative (-) Control and one preparation of the COBAS AmpliScreen HCV Positive (+) Control must be included in each A-ring (see "Quality Control" section).
2. The Specimen Preparation and Amplification Reagents are packaged in six single-use bottles. The Multiprep Negative (-) and HCV Positive (+) Controls are packaged in single-use vials. For the most efficient use of reagents, specimens and controls should be processed in batches that are multiples of 8.

B. Equipment

1. Prepare the COBAS AMPLICOR Analyzer and the Data Station for the AMPLILINK Software for use according to instructions in the *Operator's Manual* for the AMPLILINK software and the *Operator's Manual* for the COBAS AMPLICOR Analyzer.
2. Prepare the Hamilton MICROLAB AT plus 2 System and SUNPLUS Data Station or equivalent for use according to instructions in the *Operator's Manuals*.
3. Pre-cool the high-speed centrifuge and rotor to 2 - 8°C. See operating instructions for the high speed centrifuge for details.
4. Perform manufacturer recommended maintenance and calibration on all instruments, including pipettors, to ensure proper functioning.

C. Reagents

1. All reagents **except HCV MMX, v2.0 and HCV Mn²⁺, v2.0**, must be at room temperature before use. Visually examine reagents for sufficient volume before beginning the test procedure. See section "Reagent Preparation" for specific reagent storage conditions.
2. Add all reagents using a pipettor capable of delivering specified volume with ± 3% accuracy and a precision of ≤ 5% CV. Check pipettor functionality and calibrate as recommended by pipettor manufacturer.
3. Prepare Working Master Mix in a template-free area (e.g., in a dead air box). **Reagent preparation area must be clean and disinfected in accordance with methods outlined in "Precautions" (Item A). Failure to do so may result in reagent contamination.**

4. Prepare 70% ethanol fresh each day.
 5. Check expiration date of opened or Working Reagents before loading on the COBAS AMPLICOR Analyzer.
 6. Check to ensure that all reagents used are of the same master lot of kit reagents.
- D. **Workflow**
1. To minimize the possibility of laboratory areas becoming contaminated with amplicon, the laboratory area should be separated into several distinct areas organized around Pre-Amplification and Post-Amplification. Personnel should use proper anti-contamination safeguards when moving between areas.
 2. The Pre-Amplification Area should have a template-free area for preparation of Working Master Mix and an amplicon free area for specimen and control preparation.
 3. The Post-Amplification Area should have a COBAS AMPLICOR Analyzer(s) and AMPLILINK Data Station(s) with additional area for preparing Working Amplification and Detection Reagents.
 4. Pipettors and other supplies should be dedicated to a specific area. Samples, equipment and reagents should not be returned to the area where a previous step was performed.
- E. **Temperature**
- Room temperature is defined as 15° to 30°C.
- F. **Vortexing**
- Proper vortexing during sample preparation is important to ensure homogeneous mixture after additions of reagents.
- G. **Pipetting**
1. Pooled or individual plasma specimens must be at room temperature before pipetting.
 2. Use a clean pipette tip or disposable transfer pipette with each specimen or control. Use aerosol barrier or positive displacement RNase-free tips.
 3. Confirm that all pipettors are correctly set to dispense the specified volumes in accordance with the specimen preparation procedures and guidelines.
- H. **Specimen Processing**
1. Screw-cap tubes must be used for specimen and control preparation to prevent splashing and potential cross-contamination of specimens and controls. Do not use snap cap tubes.
 2. Avoid contaminating gloves when manipulating specimens.
 3. Specimens and controls should be prepared in a laminar flow hood. **Failure to do so may result in sample contamination.** Specimen and control preparation area must be cleaned and disinfected in accordance with methods outlined in "Precautions" (Item A).
- I. **Decontamination**
- Thoroughly clean and disinfect all work surfaces with a freshly prepared solution of 0.5% sodium hypochlorite in distilled or deionized water. Follow by wiping down the surface with 70% ethanol.

INSTRUCTIONS FOR USE

The Multiprep Specimen Processing Procedure is used for extracting nucleic acid from pooled specimens. The Standard Specimen Processing Procedure is used for extracting nucleic acid from individual specimens as well as for tertiary resolution.

The Multiprep and the Standard Specimen Processing Procedures are generic nucleic acid extraction procedures and can be used for the extraction of HCV RNA, HIV-1 RNA, and/or HBV DNA. A single extraction is sufficient for multiple assays. Workflow can be performed on the same day or over multiple days under the following conditions:

Amplification, Hybridization and Detection of Stored Processed Specimens

Amplification, hybridization and detection can occur on the same day as specimen processing or on a subsequent day. If amplification, hybridization and detection are to be done on a subsequent day, perform the Multiprep Specimen Processing Procedure described in steps B1 through B20 or the Standard Specimen Processing Procedure described in steps B22 through B38. Store the processed specimens and controls as indicated. On the subsequent day, begin with Step A (Reagent Preparation - Working Master Mix), thaw processed specimens and controls at room temperature, and continue with Step B39.

Hybridization and Detection of Stored Denatured Amplicon

Hybridization and detection of the denatured amplicon may occur on the same day as amplification or on a subsequent day. If hybridization and detection are to be done on a subsequent day, the denatured amplicon may be left on-board the COBAS AMPLICOR Analyzer for not more than 24 hours before starting the hybridization and detection steps. Alternatively, the denatured amplicon may be stored at 2 - 8°C for not more than five days before starting the hybridization and detection steps.

A. Reagent Preparation - Working Master Mix

Performed in: Pre-Amplification - Reagent Preparation Area (e.g., dead air box)

- A1. Determine the appropriate number of A-ring(s) needed for specimen and control testing.
- A2. Place the A-ring(s) on the A-ring holder(s).
- A3. For each A-ring, prepare one Working Master Mix.
- A4. Pipette 50 µL Working Master Mix into each A-tube. Discard unused Working Master Mix. Do not close the covers of the A-tubes at this time.
- A5. Place the A-ring containing Working Master Mix in a sealable bag and seal the plastic bag. Record the assay name (HCV) and the time the Working Master Mix was prepared.
- A6. Store the A-ring(s) containing Working Master Mix at 2 - 8°C until specimen and control preparation is completed. The A-rings with Working Master Mix must be used within 4 hours of preparation.
- A7. Decontaminate area. See "Procedural Notes", Item I.

B. Specimen and Control Preparation

Performed in Pre-Amplification - Specimen and Control Preparation Area

Multiprep Specimen Processing Procedure

- B1. Pipette 1000 µL of each pool into an appropriately labeled screw-cap tube using the COBAS AmpliScreen Pooling System, a hand-held pipettor or other user-validated method. Cap the tubes.
- B2. Vortex **NHP** briefly.
- B3. For each Negative and Positive Control pipette 1000 µL **NHP** into an appropriately labeled screw-cap tube. Cap the tubes.
- B4. Use a permanent marker to make an orientation mark on each tube.
- B5. Place the specimen and control tubes into the pre-cooled high-speed centrifuge with the orientation marks facing outward, so that the orientation marks will align with the pellets formed during centrifugation.
- B6. Centrifuge specimens and control tubes at 23,000 - 24,000 x g for 60 ± 4 minutes at 2 - 8°C. The pellet will form on the outer wall as indicated by the orientation mark.

NOTE: The 60 ± 4 minutes begins when the centrifuge reaches 23,000 - 24,000 x g.

- B7. Remove the tubes from the centrifuge and remove the caps. Slowly aspirate 900 µL of the supernatant from each centrifuged tube leaving approximately 100 µL of supernatant. Avoid contact with the pellet. Discard the supernatant and pipette tip appropriately. Use a fresh pipette tip for each tube.
- B8. Prepare a Working Lysis Reagent bottle for every batch of 8 specimens and controls to be processed.

- B9. Pipette 600 µL Working Lysis Reagent into each specimen and control tube. Cap and vortex tubes briefly.
- B10. Prepare Controls as follows:
- Negative Control
Vortex **MP (-) C** briefly. Tap vial to collect the solution in the base. Pipette 20 µL **MP (-) C** to the tube labeled "MP (-) C" containing Working Lysis Reagent and **NHP**. Cap the tube and vortex briefly.
 - Positive Control
- NOTE: A multi-analyte (HIV-1/HCV) positive control can be made by adding 20 µL of each analyte specific (+) C to the tube labeled "MP (+) C" containing Working Lysis Reagent and NHP.**
- Vortex **HCV (+) C, v2.0** and **HIV-1 (+) C, v1.5** briefly. Tap vial to collect the solution in the base. Pipette 20 µL **HCV (+) C, v2.0** and 20 µL **HIV-1 (+) C, v1.5** to the tube labeled "MP (+) C" containing the Working Lysis Reagent and **NHP**. Cap the tube and vortex briefly.
- B11. Incubate all tubes for 10 to 15 minutes at room temperature after adding Working Lysis Reagent to the last tube. After the incubation period, briefly vortex all tubes.
- B12. Pipette 700 µL of isopropanol into each tube. Cap the tubes and vortex briefly.
- B13. Place the tubes into a microcentrifuge with the orientation marks facing outward to align with the pellets that will form. Centrifuge at 14,250 ± 1750 x g for 15 - 20 minutes at room temperature.
- B14. Slowly aspirate the supernatant from each tube. Remove as much liquid as possible without disturbing the pellet.
- B15. Pipette 1.0 mL of 70% ethanol into each tube. Cap the tubes and vortex briefly.
- B16. Place the tubes into a microcentrifuge with the orientation marks facing outward to align with the pellets that will form. Centrifuge at 14,250 ± 1750 x g for 5 - 10 minutes at room temperature.
- B17. Slowly aspirate the supernatant from each tube using a fine-tip disposable transfer pipette. Remove as much liquid as possible without disturbing the pellet. Use a new transfer pipette for each tube.
- B18. Using a new transfer pipette for each tube, repeat Step B17 to remove as much of the remaining supernatant as possible without disturbing the pellet. Residual ethanol can inhibit amplification.
- B19. Pipette 200 µL **MP DIL** into each tube. Use a pipette tip to break apart the pellet. This can be done by aspirating 30-40 µL of the diluent in the tip and scraping the sides and base of the tube in an up/down motion for at least 10 seconds and dispensing 30-40 µL. Cap the tubes and vortex briefly to resuspend the extracted RNA. Note that some insoluble material may remain.
- B20. At this point amplification of the processed specimens and controls must be started within 2 hours. If not, the processed specimens and controls can be stored at -70°C or colder for up to one month. Thawing should be completed within one hour at room temperature.
- B21. Proceed to step B39, Loading the A-ring

Standard Specimen Processing Procedure

- B22. Pipette 200 µL of each specimen into an appropriately labeled screw-cap tube using the COBAS AmpliScreen Pooling System, a hand-held pipettor or other user-validated method. Cap the tubes.
- B23. Vortex **NHP** briefly.
- B24. For each Negative and Positive Control pipette 200 µL **NHP** into appropriately labeled screw-cap tubes. Cap the tubes.
- B25. Use a permanent marker to make an orientation mark on each tube.
- B26. Prepare a Working Lysis Reagent bottle for every 8 specimens and controls to be processed.
- B27. Pipette 600 µL Working Lysis Reagent into each tube. Cap and vortex tubes briefly.
- B28. Prepare Controls as follows:
- Negative Control
Vortex **MP (-) C** briefly. Tap vial to collect the solution in the base. Pipette 20 µL **MP (-) C** into the tube labeled "MP (-) C" containing Working Lysis Reagent and **NHP**. Cap the tube and vortex briefly.
 - Positive Control
- NOTE: A multi-analyte (HIV-1/HCV) positive control can be made by adding 20 µL of each analyte specific (+) C to the tube labeled "MP (+) C" containing Working Lysis Reagent and NHP.**
- Vortex **HCV (+) C, v2.0** and **HIV-1 (+) C, v1.5** briefly. Tap vial to collect the solution in the base. Pipette 20 µL **HCV (+) C, v2.0** and 20 µL **HIV-1 (+) C, v1.5** to the tube labeled "MP (+) C" containing the Working Lysis Reagent and **NHP**. Cap the tube and vortex briefly.
- B29. Incubate all tubes for 10-15 minutes at room temperature after adding Working Lysis Reagent to the last tube. After the incubation period, briefly vortex all tubes.
- B30. Pipette 800 µL of isopropanol into each tube. Cap the tubes and vortex briefly.
- B31. Place the tubes into a microcentrifuge with the orientation marks facing outward to align with the pellets that will form. Centrifuge at 14,250 ± 1750 x g for 15-20 minutes at room temperature.
- B32. Slowly aspirate the supernatant from each tube. Remove as much liquid as possible without disturbing the pellet.
- B33. Pipette 1.0 mL of 70% ethanol into each tube. Cap the tubes and vortex briefly.
- B34. Place the tubes into a microcentrifuge with the orientation marks facing outward to align with the pellets that will form. Centrifuge at 14,250 ± 1750 x g for 5 - 10 minutes at room temperature.
- B35. Slowly aspirate the supernatant from each tube using a fine-tip disposable transfer pipette. Remove as much liquid as possible without disturbing the pellet. Use a new transfer pipette for each tube.
- B36. Using a new transfer pipette for each tube, repeat Step B35 to remove as much of the remaining supernatant as possible without disturbing the pellet. **Residual ethanol can inhibit amplification.**
- B37. Pipette 200 µL **MP DIL** into each tube. Use a pipette tip to break apart the pellet. This can be done by aspirating 30-40 µL of the diluent in the tip and scraping the sides and base of the tube in an up/down motion for at least 10 seconds and dispensing 30-40 µL. Cap the tubes and vortex briefly to resuspend the extracted RNA. Note that some insoluble material may remain.
- B38. At this point amplification of the processed specimens and controls must be started within 2 hours. If not, the processed specimens and controls can be stored at -70°C or colder for up to one month. **Thawing should be completed within one hour at room temperature.**

Loading the A-ring

- B39. Create an A-ring worklist record for each A-ring to identify the A-tube with the appropriate control or specimen to be pipetted.
- B40. If processed specimens and controls were stored frozen, thaw at room temperature before proceeding. Briefly vortex the processed specimens and controls.
- B41. Pipette 50 µL of each processed specimen and control into the appropriate A-tube containing HCV Working Master Mix. Immediately cap the A-tube and repeat this step for all 8 A-tubes to complete the A-ring loading. Use the A-ring worklist record to ensure the appropriate specimen or control is added to the correct A-tube position for each A-ring.
- B42. Transfer the A-ring with sealed tubes containing the processed specimens and controls in Working Master Mix to the Amplification/Detection Area. Proceed to Part C.

NOTE: Amplification must begin within 45 minutes from when the first specimen or control in the A-ring is added to the Working Master Mix.

C. Reverse Transcription, Amplification and Detection

Performed in Post-Amplification - Amplification/Detection Area

- C1. Perform Daily Instrument Maintenance as outlined in the *Operator's Manual* for the COBAS AMPLICOR Analyzer including:
- Wipe D-cup handler tip with a lint-free moist cloth and dry.
 - Wipe initialization post with a lint-free moist cloth and dry.

- C2. Before each run:
- Check waste container and empty if necessary.
 - Check Wash Buffer Reservoir and add prepared Wash Buffer if necessary.
 - Replace used D-cup racks.
 - Prime the COBAS AMPLICOR Analyzer.
- C3. Instrument Loading and System Operation
- Prepare enough of the following detection reagent cassettes to complete the workload: Working HCV Probe Suspension Reagent (**CH2, v2.0**), Working IC Probe Suspension Reagent (**CI2**), Working Substrate (**SB1**), Denaturation Reagent (**DN2**), and Conjugate Reagent (**CN4**).
 - Place the **CH2, v2.0** and **CI2** cassettes in the test-specific reagent rack.
 - Place **DN2, CN4** and **SB1** cassettes in the generic reagent rack. Record on the cassette the date when each cassette was opened.
 - Identify the reagent racks as generic or test specific using the COBAS AMPLICOR Analyzer barcode scanner for the AMPLILINK software, as described in the *Operator's Manual* for AMPLILINK software.
 - Configure the reagent racks by entering the reagent positions and lots using the COBAS AMPLICOR Analyzer barcode scanner for the AMPLILINK software, as described in the *Operator's Manual* for AMPLILINK software.
 - Load the reagent racks onto the Analyzer using the COBAS AMPLICOR Analyzer barcode scanner for the AMPLILINK software, as described in the *Operator's Manual* for AMPLILINK software. Make sure that each reagent cassette is in its assigned position and that each cassette fits tightly into its rack.
 - Place the D-cup rack on the D-cup platform. Two D-cups are required for each A-tube and two D-cups are required for each Working Substrate cassette to allow for blanking by the COBAS AMPLICOR Analyzer, as described in the *Operator's Manual* for the COBAS AMPLICOR Analyzer.
 - Place the A-ring into the thermal cycler segment of the COBAS AMPLICOR Analyzer and close the cover on the thermal cycler segment.
 - Load the A-ring into the COBAS AMPLICOR Analyzer using the Analyzer barcode scanner for the AMPLILINK software, as described in the *Operator's Manual* for AMPLILINK software.
 - Create an A-ring order, using the AMPLILINK software, as described in the *Operator's Manual* for AMPLILINK software. Use the A-ring worklist record created for specimen processing to assist in entering the A-ring order.
 - Repeat steps h. through j. above to load a second A-ring on the COBAS AMPLICOR Analyzer.
 - Start the COBAS AMPLICOR Analyzer as described in the *Operator's Manual* for AMPLILINK software.
 - Wait for the COBAS AMPLICOR Analyzer to indicate that the load check has passed.

NOTE: The required quantity of each detection reagent is automatically calculated by the COBAS AMPLICOR Analyzer during the Load Check to determine if sufficient reagents are available for the requested tests.

- The COBAS AMPLICOR Analyzer automatically performs reverse transcription, amplification and detection. Results are expressed as absorbance values at 660 nm and as positive or negative.
- As part of an overall Quality Assurance program, if desired, for each run, print the AMPLILINK A-ring Results Report and the Run Log and retain these along with the A-ring worklist. Compare the A-ring worklist record with the AMPLILINK A-ring Results Report and verify that the A-ring ID, instrument serial number, and specimen IDs are identical. Reconcile the Run Log with the A-ring worklist to account for all A-ring IDs associated with each run. If there are discrepancies, perform follow-up investigation.

QUALITY CONTROL PROCEDURES

- At least one Multiprep (-) Control and one HCV (+) Control must be processed with each A-ring.

a. Negative Control

The absorbance for the **MP (-) C** should be less than 0.1 at 660 nm and its associated **MP IC** should be greater than or equal to 0.2 at 660 nm for the Negative Control to be valid. If the absorbance value for the **MP (-) C** is greater than or equal to 0.1 and/or its associated **MP IC** is less than 0.2, the entire A-ring is invalid, and the entire test procedure for that A-ring (specimen and control preparation, amplification and detection) must be repeated.

b. Positive Control

The absorbance for the **HCV (+) C, v2.0** should be greater than or equal to 1.0 at 660 nm and its associated **MP IC** should be greater than or equal to 0.2 at 660 nm for the Positive Control to be valid. If the absorbance value for the **HCV (+) C, v2.0** is less than 1.0 and/or its associated **MP IC** is less than 0.2, the entire A-ring is invalid, and the entire test procedure for that A-ring (specimen and control preparation, amplification and detection) must be repeated.

Summary of Control Acceptance Criteria

	HCV Result		IC Result	
	A ₆₆₀	Comment	A ₆₆₀	Comment
Negative Control	< 0.1	Negative	≥ 0.2	Valid
Positive Control	≥ 1.0	Positive	≥ 0.2	Valid

- Flags and comments may be generated by the COBAS AMPLICOR Analyzer during a run. The Operator must check the run printout(s) for flags and comments to verify that the run is valid. Refer to the *Operator's Manual* for the AMPLILINK software and the *Operator's Manual* for the COBAS AMPLICOR Analyzer for interpretation of flags and comments.
- External Control
If required by the laboratory or by a regulatory agency, an External Run Control must be processed in the same manner as a sample. The External Control should contain a defined number of target sequence copies and the level of this control should be a multiple of the cut-off value of the test system. The absorbance of the HCV External Run Control should be equal to or greater than 0.2 at 660 nm. Any absorbance value for **MP IC** is acceptable. If the absorbance of the HCV External Run Control does not meet the above criteria, the negative samples may be in question. Therefore, the laboratory should follow their established Standard Operating Procedure for the appropriate action.

INTERPRETATION OF RESULTS

- Flags and comments may be generated by the COBAS AMPLICOR Analyzer during a run. The Operator must check the run printout(s) for flags and comments to verify that the run is valid. Refer to the *Operator's Manual* for the AMPLILINK software and the *Operator's Manual* for the COBAS AMPLICOR Analyzer for interpretation of flags and comments.
- Specimen Results
Two absorbance values are obtained for each specimen: one for the HCV target and one for the internal control (**MP IC**). For a sample with an absorbance less than 0.2, the **MP IC** absorbance for that specimen must be greater than or equal to 0.2 at 660 nm for a valid negative specimen test result. If the absorbance for the HCV target is greater than or equal to 0.2, the **MP IC** result is disregarded and the test result is valid and positive.

3. For a valid run, results are interpreted as follows:

HCV Result		IC Result		Interpretation
A ₆₆₀	Comment	A ₆₆₀	Comment	
< 0.2	NEGATIVE	≥ 0.2	VALID	Specimen is negative for HCV RNA.
< 0.2	NEGATIVE	< 0.2	INVALID	Invalid result. Repeat entire test procedure for invalid specimen.
≥ 0.2	POSITIVE	ANY	VALID	Specimen is positive for HCV RNA.

Invalid Test Runs

When invalid Positive or Negative Control results are obtained on an A-ring, that A-ring is invalid. Repeat the entire test procedure for the associated specimens (including specimen and control preparation, amplification and detection) in the A-ring by processing another aliquot of the original plasma specimens.

With the exception of instrument failures subsequent to denaturation of amplicon, an instrument failure during a test run, as indicated by system error messages, also constitutes an invalid test run. In such instances, repeat the test procedure for the associated controls and specimens (amplification and detection) in the run by processing another aliquot of the processed specimen.

For instrument failures subsequent to successful denaturation of amplicon, it is not necessary to repeat the entire test procedure for the associated specimens. In such instances, the denatured amplicon may be redetected by the COBAS AMPLICOR Analyzer. The denatured amplicon may be left on the COBAS AMPLICOR Analyzer for not more than 24 hours before continuing with the hybridization and detection steps. Alternatively, the denatured amplicon may be stored at 2 - 8°C for not more than five days before continuing with the hybridization and detection steps.

Invalid Specimen Results

For specimen(s) that are invalid, perform repeat testing in single on the remaining replicate tube(s). The test result for the pool or individual donor specimen is based only on the repeat valid test result. If the last available replicate of a pooled specimen gives an invalid result, each individual donor specimen in that pool should be tested. If an individual donor specimen gives an invalid result, the test result for that individual donor specimen should be considered invalid for HCV RNA.

Results of Pooled Donor Specimens

Testing of pooled samples for the COBAS AmpliScreen HCV Test, v2.0 requires a single level of testing for Primary Pools that are negative for HCV RNA and three levels of testing (Primary Pool, Secondary Pool and tertiary resolution) for Primary Pools that are positive for HCV RNA.

Negative Primary Pools

When the Primary Pool is negative, report the results for all associated individual donor specimens in that Primary Pool as "HCV RNA Negative".

Positive Primary Pools - Secondary Pool Testing

When the Primary Pool is positive, prepare four Secondary Pools containing the associated donor specimens. The Secondary Pools must be processed using the Multiprep Specimen Processing Procedure.

- If one or more of the Secondary Pools tests positive, report the results for the donor specimens in the negative Secondary Pools as "HCV RNA Negative". For positive Secondary Pools, proceed to the section entitled "**Positive Primary Pool, Positive Secondary Pools - Tertiary Resolution Testing.**"
- If all four Secondary Pools are negative, the individual donor specimens in that Primary Pool may be reported as "HCV RNA Negative."
- As part of an overall Quality Assurance program, you may wish to conduct additional testing to determine the cause of the initial positivity of the Primary Pool, for which the four associated secondary pools are negative. Positive pools resolved as negative at the secondary resolution testing level is usually a result of a contamination event to the primary pool, but theoretically may be due to a viral load below the limit of detection.

Positive Primary Pool, Positive Secondary Pools - Tertiary Resolution Testing

For a positive Secondary Pool, test each of the individual donor specimens in that Secondary Pool. The individual donor specimens must be processed using the Standard Specimen Processing procedure.

- If one or more of the individual donor specimens is positive, the positive donor specimen(s) is (are) reported as "HCV RNA Positive" and the remaining negative donor specimens associated with the positive Secondary Pool are reported as "HCV RNA Negative."
- If all of the individual donor specimens in that Secondary Pool test negative, the donor specimens in the Secondary Pool may be reported as "HCV RNA Negative."
- As part of an overall Quality Assurance program, you may wish to conduct additional testing to determine the cause of the positivity of the Primary and Secondary Pools, for which the associated individual samples were negative. Positive pools resolved as negative at the tertiary resolution testing level is usually a result of a contamination event to the secondary pool, but theoretically may be due to a viral load below the limit of detection.

Results of Individual Donor Samples

If an individual donor specimen is positive, the positive donor specimen is reported as "HCV RNA Positive."

If an individual donor specimen is negative, the negative donor specimen is reported as "HCV RNA Negative."

PROCEDURAL LIMITATIONS

1. This test has been evaluated only for use in combination with the COBAS AmpliScreen Multiprep Specimen Preparation Reagents Kit, COBAS AmpliScreen Negative Human Plasma Reagent Kit, COBAS AMPLICOR Analyzer and the Hamilton MICROLAB AT plus 2 Pipettor for the automated preparation of plasma pools.
2. **Heparin inhibits PCR; specimens collected using heparin as the anticoagulant should not be used with the COBAS AmpliScreen HCV Test, v2.0.**
3. Reliable results are dependent on adequate specimen collection and proper transport procedures.
4. Detection of HCV RNA is dependent on the number of virus particles present in the specimen and may be affected by specimen collection methods, patient factors (i.e., age, presence of symptoms), and/or stage of infection and pool size.
5. Only the Hamilton MICROLAB AT plus 2 Pipettor has been validated for use with the COBAS AmpliScreen HCV Test, v2.0 for the automated preparation of plasma pools. Adhere to the hardware instructions and safety precautions outlined in the User Manual for the Hamilton MICROLAB AT plus 2 Pipettor. For detailed instructions on the use of the Hamilton MICROLAB AT plus 2 pipettor for the COBAS AmpliScreen Test, contact your local Roche representative to obtain a copy of the COBAS AmpliScreen Pooling System Guide.

PERFORMANCE CHARACTERISTICS**Reproducibility**

The reproducibility of the Test was established by testing two six-member EDTA plasma panels with known concentrations of HCV. Panel One, which was tested using the Multiprep Specimen Processing Procedure, contained one HCV-negative sample and HCV-positive samples with HCV RNA concentrations of 10, 25, 50, and 50,000 IU/mL. Panel Two, which was tested using the Standard Specimen Processing Procedure, contained one HCV-negative sample and HCV-positive samples with concentrations of 25, 50, 100 and 50,000 IU/mL.

Testing was performed at three sites with two operators at each site using three COBAS AmpliScreen HCV Test, v2.0 kit lots. Each operator used a dedicated COBAS AMPLICOR Analyzer throughout the study. Each operator was provided panel sets that had been randomized and labeled in blinded fashion.

All valid reproducibility data were evaluated by calculating the percentage of correct results for each panel member. The data were analyzed by site, lot, testing day, run, and operator for each Specimen Processing Procedure (Multiprep and Standard).

The reproducibility study for the COBAS AmpliScreen HCV Test, version 2.0 demonstrated consistency by lot and site for both the Multiprep and Standard Specimen Processing Procedures as seen in Tables 1 and 2 below:

Table 1
Reproducibility Results- Multiprep Specimen Processing Procedure

Results By Lot (Number Positive/Number Tested)					
	Negative	10 IU/mL	25 IU/mL	50 IU/mL	50,000 IU/mL
Lot #1	0/89	72/89	164/177	88/90	90/90
(%)	(0%)	(81%)	(93%)	(98%)	(100%)
Lot #2	0/90	59/90	168/180	88/89	90/90
(%)	(0%)	(66%)	(93%)	(99%)	(100%)
Lot #3	0/90	59/90	170/179	88/89	90/90
(%)	(0%)	(66%)	(95%)	(99%)	(100%)
Results By Site (Number Positive/Number Tested)					
Site #1	0/90	66/89	166/178	88/89	90/90
(%)	(0%)	(74%)	(93%)	(99%)	(100%)
Site #2	0/89	65/90	170/179	90/90	90/90
(%)	(0%)	(72%)	(95%)	(100%)	(100%)
Site #3	0/90	59/90	166/179	86/89	90/90
(%)	(0%)	(66%)	(93%)	(97%)	(100%)

Table 2
Reproducibility Results- Standard Specimen Processing Procedure

Results By Lot (Number Positive/Number Tested)					
	Negative	25 IU/mL	50 IU/mL	100 IU/mL	50,000 IU/mL
Lot #1	0/90	56/89	166/180	89/90	90/90
(%)	(0%)	(63%)	(92%)	(99%)	(100%)
Lot #2	0/90	66/89	165/179	89/90	90/90
(%)	(0%)	(74%)	(92%)	(99%)	(100%)
Lot #3	3/87	68/90	167/179	83/87	90/90
(%)	(3%)	(76%)	(93%)	(95%)	(100%)
Results By Site (Number Positive/Number Tested)					
Site #1	0/87	61/89	162/179	85/87	90/90
(%)	(0%)	(69%)	(91%)	(98%)	(100%)
Site #2	1/90	72/90	169/179	88/90	90/90
(%)	(1%)	(80%)	(94%)	(98%)	(100%)
Site #3	2/90	57/89	167/180	88/90	90/90
(%)	(2%)	(64%)	(93%)	(98%)	(100%)

Analytical Sensitivity - Dilutional Panels

The analytical sensitivity of the COBAS AmpliScreen HCV Test, v2.0 was determined by testing 10 HCV seropositive clinical specimens. The titer of each specimen was quantitated with a commercially available assay using a secondary standard calibrated against the WHO International Standard. These specimens were diluted in normal human plasma to 150, 50, 16.7 and 5.6 HCV RNA IU/mL for the Multiprep Specimen Processing Procedure and 300, 100, 33.3 and 11.1 IU/mL for the Standard Specimen Processing Procedure. The COBAS AmpliScreen HCV Test, v2.0 detected 16.7 HCV RNA IU/mL at a frequency greater than 90% with a lower 95% confidence limit of 86.4% using the Multiprep Specimen Processing Procedure. The assay detected 33.3 HCV RNA IU/mL at a frequency greater than 84% with a lower 95% confidence limit of 79.7% using the Standard Specimen Processing Procedure. The data are presented in Tables 3 and 4.

When evaluated using PROBIT analysis, the combined data for all samples processed by the Multiprep Specimen Processing Procedure indicate an average 95% Limit of Detection (LOD) of 21.0 IU/mL, with lower and upper 95% confidence limits of 17.1 IU/mL and 27.8 IU/mL, respectively. The LOD of 21.0 IU/mL corresponds to approximately 57 copies/mL. This equates to an LOD of 504 IU/mL or 1368 copies/mL for an individual donor specimen tested in a pool of 24 donors.

When evaluated using PROBIT analysis, the combined data for all samples processed by the Standard Specimen Processing Procedure indicate an average 95% LOD of 54.1 IU/mL, with lower and upper 95% confidence limits of 44.1 IU/mL and 71.7 IU/mL, respectively. The LOD of 54.1 IU/mL corresponds to approximately 146 copies/mL.

Table 3
Multiprep Procedure Testing Summary for All Clinical Samples
Combined Input Values with 95% One-tailed Lower Confidence Limit

Multiprep Sample Processing Procedure				
HCV RNA Concentration (IU/mL)	Number of Positives	Number of Individual Trials	% Positive	95% Lower Confidence Limit - One-Tailed
150	219	219	100.0%	98.6%
50	220	220	100.0%	98.6%
16.7	197	218	90.3%	86.4%
5.6	30	44	68.1%	54.8%

Table 4
Standard Procedure Testing Summary for All Clinical Samples
Combined Input Values with 95% One-tailed Lower Confidence Limit

Standard Sample Processing Procedure				
HCV RNA Concentration (IU/mL)	Number of Positives	Number of Individual Trials	% Positive	95% Lower Confidence Limit – One-Tailed
300	220	220	100.0%	98.6%
100	220	220	100.0%	98.6%
33.3	183	217	84.3%	79.7%
11.1	54	87	62.1%	52.7%

Analytical Sensitivity - WHO HCV International Standard

The analytical sensitivity of the COBAS AmpliScreen HCV Test, v2.0 was also determined using the WHO HCV International Standard (96/790). The WHO HCV International Standard was serially diluted in HCV-negative plasma to final concentrations of 200, 100, 50, 25, 15, and 10 IU/mL. Each dilution was tested with two lots of the COBAS AmpliScreen HCV Test, v2.0 using both the Multiprep and Standard Specimen Processing Procedures.

When evaluated using PROBIT analysis, the combined data for all samples processed by the Multiprep Specimen Processing Procedure indicate an average 95% LOD of 28.8 IU/mL, with lower and upper 95% confidence limits of 20.5 IU/mL and 85.8 IU/mL, respectively. This equates to an LOD of 691.2 IU/mL for an individual donor specimen tested in a pool of 24 donors.

When evaluated using PROBIT analysis, the combined data for all samples processed by the Standard Specimen Processing Procedure indicate an average 95% LOD of 41.9 IU/mL, with lower and upper 95% confidence limits of 28.0 IU/mL and 111.8 IU/mL, respectively. Tables 5 and 6 summarize the overall results for the Multiprep and Standard Specimen Processing Procedures, respectively.

Table 5
Serial Dilution Testing Summary for Multiprep Method
Combined Input Values with Lower 95% Confidence Limit (One-Sided)

HCV RNA Concentration (IU/mL)	Number of Positives	Number of Individual Trials	% Positive	95% Lower Confidence Limit (One-sided)
200	132	132	100.00%	97.76%
100	132	132	100.00%	97.76%
50	130	132	98.48%	95.31%
25	128	132	96.97%	93.20%
15	95	132	71.97%	64.83%
10	92	132	69.70%	62.45%

Table 6
Serial Dilution Testing Summary for Standard Method
Combined Input Values with Lower 95% Confidence Limit (One-Sided)

HCV RNA Concentration (IU/mL)	Number of Positives	Number of Individual Trials	% Positive	95% Lower Confidence Limit (One-sided)
200	131	131	100.00%	97.74%
100	129	132	97.73%	94.23%
50	132	132	100.00%	97.76%
25	115	132	87.12%	81.31%
15	93	131	70.99%	63.77%
10	84	132	63.64%	56.19%

Analytical Sensitivity - CBER HCV Panel

The FDA CBER HCV Panel Members # 1-10 were processed using the Multiprep and Standard Sample Processing Procedures. Both specimen processing methods detected HCV RNA at 50 copies/mL. The Multiprep Sample Processing Procedure detected 100% of all positive members ranging from 10 - 100,000 copies/mL. The Standard Sample Processing Procedure detected 100% of all positive members ranging from 50 to 100,000 copies/mL. Both negative members of the panel were negative by both methods. The data are shown in Table 7.

Table 7
CBER HCV RNA Panel Results

CBER HCV RNA Panel (Copies/mL)	CBER HCV Panel Member Test Results (Percent Positive)									
	1 (1000)	2 (Neg)	3 (100,000)	4 (10,000)	5 (Neg)	6 (500)	7 (200)	8 (50)	9 (10)	10 (5)
Multiprep Method	100%	0%	100%	100%	0%	100%	100%	100%	100%	67%
Standard Prep Method	100%	0%	100%	100%	0%	100%	100%	100%	67%	0%

Genotype Detectability

Twenty individual plasma specimens representing Genotypes 1 and 4, sixteen plasma specimens of Genotype 2, nineteen plasma specimens of Genotype 3, and two plasma specimens each of Genotypes 5 and 6 were tested. With the exception of one sample (Genotype 2a/2c), which was below the limit of quantitation by a quantitative assay, each specimen was diluted to approximately 200 IU/mL of HCV RNA in pooled negative human plasma. Diluted samples were processed using both the Multiprep and Standard Sample Processing Procedures. The COBAS AmpliScreen HCV Test, v2.0 detected all Genotypes at 200 IU/mL except the one sample that was not quantifiable. This sample (Genotype 2a/2c) was detected using the Multiprep Specimen Processing Procedure, but was negative when tested using the Standard Specimen Processing Procedure. This result is consistent with HCV RNA levels below the detection limit of the assay. Data are provided in Table 8.

Table 8
HCV Genotype Samples Tested

HCV Genotype/Subtype	Quantity	Reactive / Total (Multiprep)	Reactive / Total (Standard Prep)
1	8	8/8	8/8
1a	3	3/3	3/3
1b	9	9/9	9/9
2	1	1/1	1/1
2a	2	2/2	2/2
2b	10	10/10	10/10
2a/2c	3	3/3	2/3*
3a	12	12/12	12/12
3a	6	6/6	6/6
3e	1	1/1	1/1
4	1	1/1	1/1
4	11	11/11	11/11
4a	2	2/2	2/2
4c	3	3/3	3/3
4c/4d	2	2/2	2/2
4h	1	1/1	1/1
5a	2	2/2	2/2
6a	2	2/2	2/2

* One sample contained HCV RNA at a level below the Limit of Quantitation of a quantitative assay. Sample was tested undiluted.

Seroconversion Panels

Nine anti-HCV seroconversion panels were tested using both the Multiprep and the Standard Specimen Processing Procedures. Each specimen in each panel was tested by the Ortho HCV, version 3.0 ELISA Test system and all samples with reactive EIA results were also tested by Chiron RIBA HCV 3.0 SIA. The HCV RNA test results were then compared to the EIA test results for each specimen to determine if HCV RNA testing detected the presence of HCV infection prior to seroconversion.

The COBAS AmpliScreen HCV Test, v2.0 detected HCV infection before seroconversion for nine seroconversion panels. The data are summarized in Table 9.

Table 9
HCV Seroconversion Study

Panel	Day Positive Ortho 3.0 EIA and Chiron RIBA 3.0	Day Positive COBAS AmpliScreen v2.0	Difference COBAS AmpliScreen vs EIA
6212	14	0	14
6224	19	0	19
6215	20	0	20
9047	28	0	28
9045	41	0	41
6225	78	39	39
6213	43	11	32
6222	40	17	23
6227	74	0*	74*

* Specimen was RNA positive on Day 0, but negative on Days 22 and 24. Day 74 specimen was RNA positive again.

Analytical Specificity - Potentially Cross Reactive and Interfering Microorganisms

The analytical specificity of the COBAS AmpliScreen HCV Test, v2.0 was evaluated by testing a panel of microorganisms and other disease states, including 23 viral isolates, two bacterial strains and one yeast isolate. No-cross reactivity was observed with the COBAS AmpliScreen HCV Test, v2.0. Table 10 summarizes the microorganisms studied.

Table 10
Analytical Specificity - Microorganisms Tested

Adenovirus type 2	Epstein Barr Virus	HIV-1 Subtype D
Adenovirus type 3	Hepatitis A Virus	HIV-2
Adenovirus type 7	Hepatitis B Virus (n=3)	HTLV-1
Autoimmune samples	Herpes Simplex type 1	HTLV-II
Candida albicans	Herpes Simplex type 2	Human Herpes Virus 6
Chlamydia trachomatis	HIV-1 Subtype A	Human Herpes Virus 7
Coxsackievirus B1	HIV-1 Subtype B	Staphylococcus epidermidis
Cytomegalovirus	HIV-1 Subtype C	Varicella-Zoster
Echovirus 1		

Up to ten individual patient plasma specimens from each of the following disease categories were spiked with low levels of HCV-positive plasma (within 2-3X the 95% LOD): HIV-1, HIV-2, autoimmune disease, EBV, CMV, and *Candida albicans*. No false negative test results were observed.

Analytical Specificity - Non-HCV Hepatitis Samples

Twenty-five HAV- and 25 HBV-positive specimens (all HCV-negative) were tested for cross reactivity with the COBAS AmpliScreen HCV Test, v2.0 by using both the Standard and Multiprep Sample Processing Procedures. All samples were found to be negative. No false positive test results were observed.

These samples were also spiked with low levels of HCV-positive plasma and tested using both the Standard and Multiprep Sample Processing Procedures. All samples were found to be positive. No false negative test results were observed.

Potentially Interfering Substances

Endogenous Interfering Substances

HCV-spiked and non-spiked plasma samples derived from whole blood containing abnormally high concentrations of bilirubin (up to 20 mg/mL), triglycerides (up to 3000 mg/dL), hemoglobin (up to 1.0 g/dL), and albumin (up to 6 g/dL) were tested. These endogenous substances did not interfere with the sensitivity or specificity of the COBAS AmpliScreen HCV Test, v2.0, using either the Standard or Multiprep Specimen Processing Procedure.

Exogenous Interfering Substances

HCV-spiked and non-spiked plasma samples derived from whole blood containing abnormally high concentrations of aspirin (up to 50 mg/mL), pseudoephedrine-HCl (up to 3 mg/dL), ascorbic acid (up to 20 mg/dL), acetaminophen (up to 40 mg/dL), or ibuprofen (up to 40 mg/dL) were tested. These exogenous substances did not interfere with the sensitivity or specificity using either the Standard or Multiprep Specimen Processing Procedure.

REFERENCES

1. Choo, Q-L., Weiner, A.J., Overby, L.R. et al. 1990. Hepatitis C Virus: The major causative agent of viral non-a, non-b hepatitis. *British Medical Bulletin* **46**:423-441.
2. Alter, H. 1991. Descartes before the horse: I clone, therefore I am: The hepatitis C virus in current perspective. *Annals of Internal Medicine* **115**:644-649.
3. Dodd RY. 1994 Adverse consequences of blood transfusion: quantitative risk estimates. In: Nance ST, ed. *Blood supply: risks, perceptions and prospects for the future*. Bethesda: American Association of Blood Banks 1-24.
4. Schreiber GB, Busch MP, Kleinman SH, Korelitz JJ. 1996 The risk of transfusion-transmitted viral infections. *The Retrovirus Epidemiology Donor Study*. *N Engl J Med* **334**:1685-90.
5. Holland PV. 1996 Viral infections and the blood supply (editorial). *N Engl J Med* **334**:1734-35.
6. Kleinman SH, Busch MP. General overview of transfusion transmitted infections. In: Petz LD, Swisher S, Kleinman SH, Spence R, Strauss RG, eds. *1995 Clinical practice of transfusion medicine*, 3rd ed. New York: Churchill Livingstone 809-21.
7. Busch MP, Stramer SL, Kleinman SH. 1997 Evolving applications of nucleic acid amplification assays for prevention of virus transmission by blood components and derivatives. In: Garratty G, ed. *Applications of molecular biology*. Bethesda, American Association of Blood Banks 121-73. (presented at a workshop during the 50th Annual Meeting of the AABB, October 1997, Denver, CO).
8. Soriano V, Dronda F, Gonzalez-Lopez A et al. 1994 HIV-1 causing AIDS and death in a seronegative individual [letter]. *Vox Sang* **67**:410-11.
9. Centers for Disease Control and Prevention. Persistent lack of detectable HIV-1 antibody in a person with HIV infection - Utah, 1995. *MMWR* 1996 **45**:181-85.
10. Myers, T.W. and Gelfand, D.H. 1991. Reverse transcription and DNA amplification by a *Thermus thermophilus* DNA polymerase. *Biochemistry* **30**:7661-7666.
11. Longo, M.C., Berninger, M.S. and Hartley, J.L. 1990. Use of uracil DNA glycosylase to control carry-over contamination in polymerase chain reactions. *Gene* **93**:125-128.
12. Richmond, J.Y. and McKinney, R.W. eds. 1999 *Biosafety in Microbiological and Biomedical Laboratories*. HHS Publication Number (CDC) 93-8395.
13. NCCLS. Protection of Laboratory Workers from Infectious Disease Transmitted by Blood, Body Fluids, and Tissue. Approved Guideline. NCCLS Document M29-A Villanova, PA:NCCLS, 1997.
14. International Air Transport Association. *Dangerous Goods Regulations*, 41st Edition. 2000. 704 pp.



Roche Molecular Systems, Inc., Branchburg, NJ 08876 USA
A Member of the Roche Group



Distributed by

Roche Diagnostics Corporation
Indianapolis, IN 46256 USA
(For Technical Assistance call the
Roche Response Center®
toll-free: 1-800 526 1247)

Roche Diagnostics (Schweiz) AG
CH-6343 Rotkreuz

Roche Diagnostics GmbH
D-68298 Mannheim, Germany

Roche Diagnostics S.L.
E-08006 Barcelona

Roche Diagnostics
Laval, Quebec, Canada H7V 4A2
(For Technical Assistance call:
Pour toute assistance technique,
appeler le: 1-877 273 3433)

Roche Diagnostics
F-38240 Meylan

Distributore in Italia:
Roche Diagnostics SpA
Piazza Durante 11
I-20131 Milano

Distribuidor em Portugal:
Roche Farmacêutica Química, Lda
P-2700 Amadora

United States:

Roche® is a registered trademark of Hoffmann-La Roche Inc.

AmpErase® and AMPLILINK® are registered trademarks and COBAS AMPLICOR™ and COBAS AmpliScreen™ are trademarks of Roche Molecular Systems, Inc., licensed to Roche Diagnostics Corporation.

Roche Response Center® is a registered service mark of Roche Diagnostics Corporation.

Other Countries:

The above trademarks are either owned by or used under license by the Roche affiliate distributing this product.

Dynabeads® paramagnetic particles are licensed under patents owned by Dynal Biotech ASA, Oslo, Norway. Dynabeads® is a registered trademark of Dynal Biotech ASA, Oslo, Norway, licensed to Roche Diagnostics Corporation, Indianapolis, Indiana.

Eppendorf®, Eppendorf Multipette® and Eppendorf Combitip® are registered trademarks of Eppendorf-Netheler-Hinz GmbH, Hamburg, Germany.

MicroAmp® is a registered trademark of Applied Biosystems Corporation or its subsidiaries in the U.S. and certain other countries.

MICROLAB® is a registered trademark of the Hamilton Company.

ProClin® is a registered trademark of Rohm and Haas Company.

Copyright 2005, Roche Molecular Systems, Inc. All rights reserved.

7/2005

00058004092-06



Roche Diagnostics GmbH
D-68298 Mannheim

